



## Manual: Trionic Cable Lock



① Locate the screw on the bracket.



② Unscrew the screw with a 4 mm allen key.



③ Pull the end of the strap out of the bracket.



④ Wrap the strap around the Veloped rear vertical frame tube (on the Walker, the semi-vertical frame tube).




⑤ Insert the end of the strap into the bracket.



⑥ Tighten the screw/strap with the 4 mm allen key until the strap is tight.



⑦ Cut off the excessive length of the strap with a cable cutter or knife.

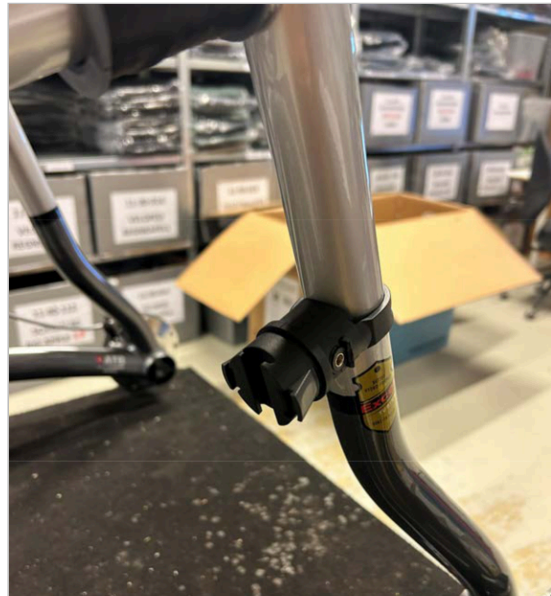

⑧ The photo shows the correct attachment of the bracket.



⑨ Attach the cable lock to the bracket from above, push the cable lock downwards until it clicks into the bracket.



⑩ To detach the cable lock from the bracket, push the two grey buttons and remove the lock from the bracket.