




## Veloped Bottle Holder – Assembly Instructions


Veloped bottle holder, that is fastened to the Veloped frame with a screwed clamp. Color: black.

The delivery includes a bottle holder, a clamp mount and a 3 mm Allen key.


Scan the QR-code with your smart phone or device to view video instructions.

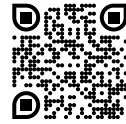

① Unscrew and remove one screw from the clamp mount.

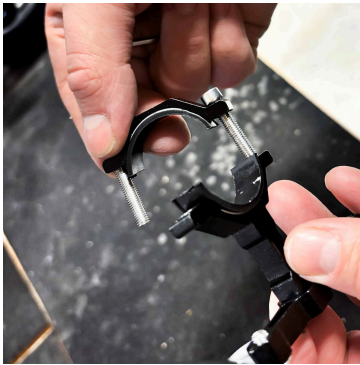

② Unscrew the other screw as far as possible, but leave it attached to the clamp.

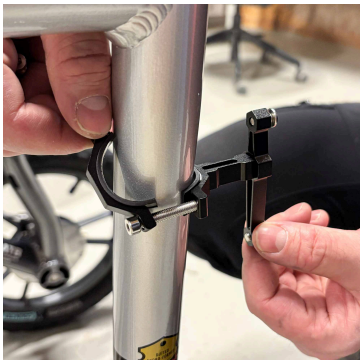

③ Rotate one half of the clamp 90 degrees to make it easier to mount the clamp on the Veloped frame. With the short arm of the bottle holder upwards, attach the clamp to the vertical part of the right or left rear frame tube of the Veloped. When screwing the clamp onto the Veloped frame, make sure that both screws are tightened equally.



④ Loosen the two screws on the clamp bracket and then screw the bottle holder onto it, preferably with the side opening of the bottle holder facing outwards.



⑤ The Veloped Bottle Holder is now correctly mounted and ready for use.