



## **Priručnik: Zamjena remena naslona za hodalicu Trionic**




① Odvrnite vijak koji drži remen pomoću imbus ključa od 3 mm.

Ponovite na suprotnoj strani.



② Obavezno izvadite vijak, maticu i cjevasti odstožnik.

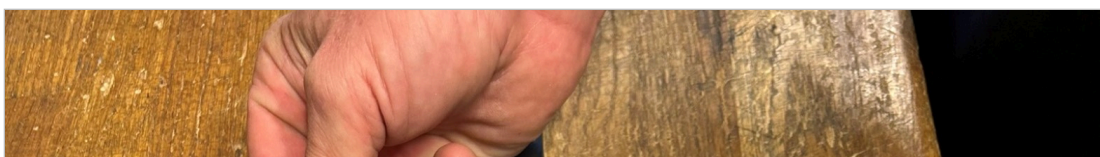



## Priručnik: Zamjena remena naslona za hodalicu Trionic



- ⑤ Umetnite novi zamjenski remen u podstavu.

**VAŽNO:** Obavezno poravnajte kraj remena s logotipom TRIONIC kao što je prikazano na fotografiji.



- ⑥ Umetnite cjevasti odstojnik u otvor na kraju remena.

